




Face Mask Instructions




Wash your hands before touching the mask

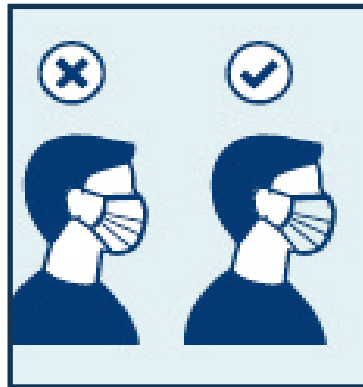

Ensure the proper side of the mask faces outward



Secure the strings behind your ears.

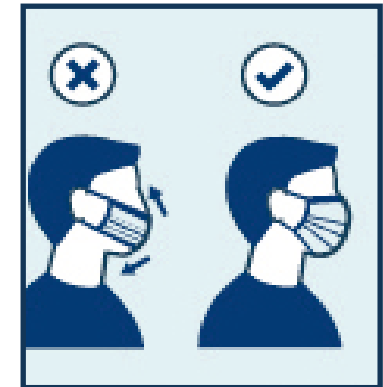

Cover mouth and nose fully making sure there are no gaps.




Do not touch the mask while using it, if so wash your hands



Remove the mask from behind by holding the strings, with clean hands

When washing your mask make sure its at 60c + do not use bleach.

When washed and tumble dried ensure you store it in a sealed plastic bag until you wish to remove to wear.

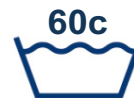

Machine wash



Tumble dry



Do not use fabric softener



Do not use bleach

UK Shop Local Ltd, www.shoplocaluk.org